

New!


UNI-GRIP® 360

UNIVERSAL SENSOR HOLDER

All you need is one!



Adjustable bite-tab allows you to take a full mouth series with only one holder.

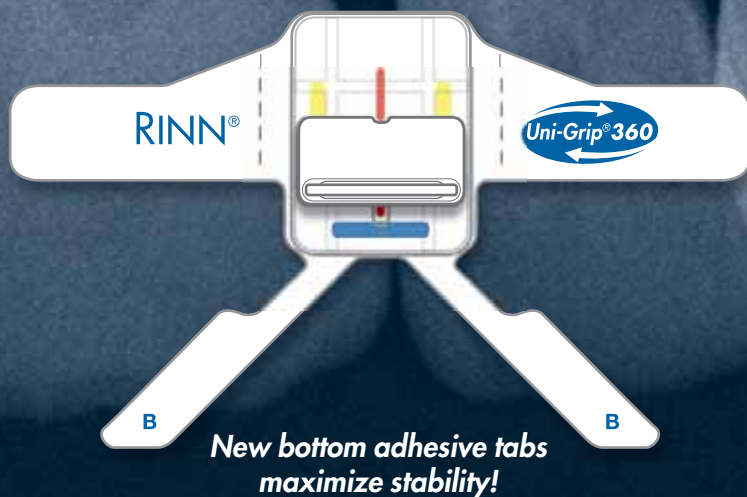

The *Improved* Uni-Grip 360 Universal Disposable Sensor Holder

Save time during your digital radiograph procedures with the new Uni-Grip 360 Universal Disposable Sensor Holder.

Improved to make digital radiographs easy and accurate:

- 70% more positioning range in posterior and 25% in anterior indications to help capture the anatomy you want to see
- 20% thinner base means less bulk — Patient friendly!
- Stiffer bite-tab for less flexing
- Perforated adhesive tabs for fast and easy removal
- Single-use for optimal patient protection
- Economical vs. different holders for each indication
- Use with Uni-GripAR 3-in-1 arm & ring positioning system for stress-free paralleling

Special Offers!

BUY 4 KITS, GET 1 FREE!

Mix & Match. Value of free kit equal to least expensive kit purchased.

PROMO CODE RRR00000*

BUY 1 50-PACK, GET A FREE UNI-GRIPAR®

\$46.75 Value

PROMO CODE RRR00000*

- 550052 Uni-Grip® 360 Kit
Package of 50 Uni-Grip 360 Universal Sensor Holders & 50 Sensor Covers MSRP \$57.95
- 550252 Uni-Grip® 360 Combo Pack
Package of 100 Uni-Grip 360 Universal Sensor Holders, 100 Sensor Covers, and 1 Uni-GripAR Positioning Arm and Ring MSRP \$120.98
- 550100 Uni-GripAR® Positioning Arm and Ring MSRP \$46.75
- 550500 Universal Sensor Covers, pkg. of 500 MSRP \$53.75

For better dentistry

DENTSPLY
RINN


www.RinnCorp.com
www.UniGrip.com
800-323-0970

UNI-GRIP[®] 360

UNIVERSAL SENSOR HOLDER


The ideal solution for fast, easy and accurate intraoral radiographs.

Uni-Grip 360 is easily applied. The four adhesive tabs keep it stable without need for tight sensor covers.

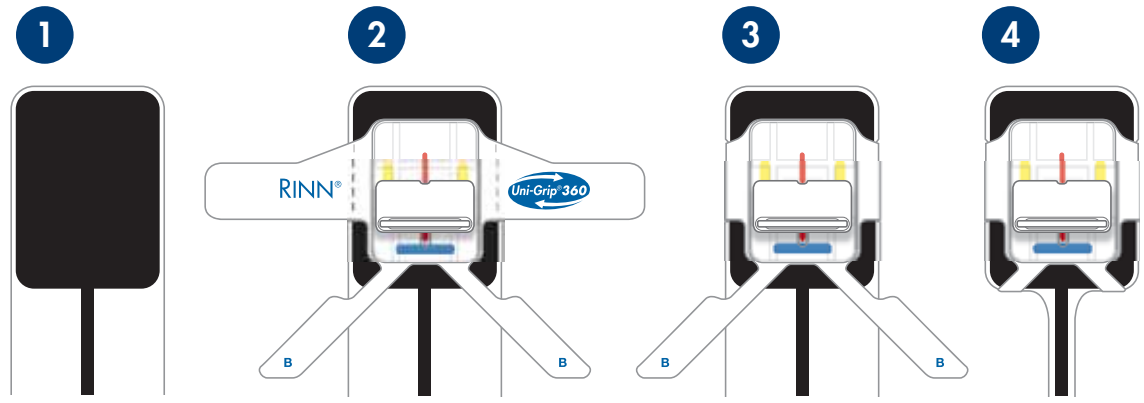

Place a protective cover over your sensor. Apply Uni-Grip 360 by removing adhesive liner and placing over the center of your sensor's active area. Wrap side and bottom tabs tightly around covered sensor. Each Uni-Grip 360 package comes with universal sensor covers for your convenience.

The all new Uni-Grip 360 bite-tab can be rotated anywhere on the sensor base to achieve the desired indication. The bite-tab can extend past the edge of the Uni-Grip 360 base to the position that is ideal for your sensor's active area.

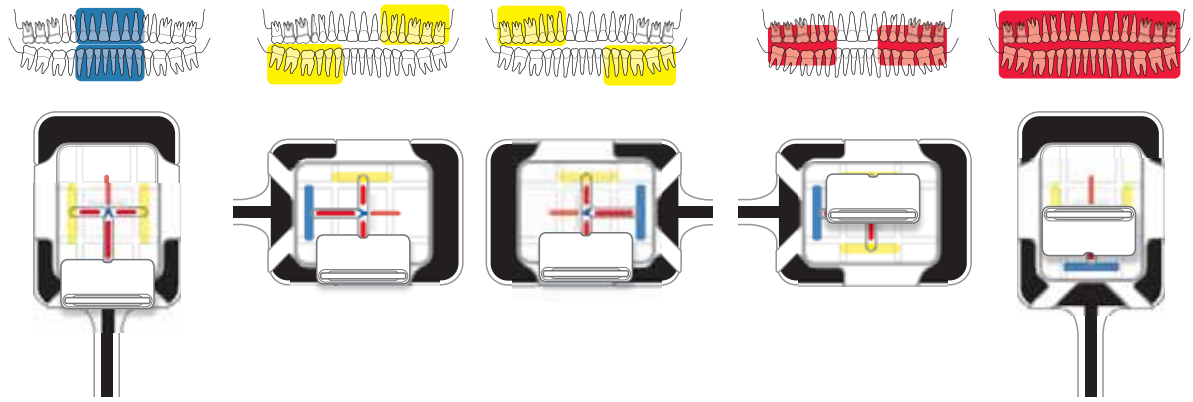

Perforated side labels make removal easy.



Uni-Grip 360 can be used with the Uni-GripAR Arm and Ring positioning system for more accurate positioning.



Need assistance? Ask RINN at 800-323-0970.

DENTSPLY
RINN

A Division of
DENTSPLY International Inc.
1212 Abbott Drive
Elgin, IL 60123 USA

RINN Customer Service:
800-323-0970 USA
847-742-1140 International
www.RinnCorp.com
www.UniGrip.com



Patent Pending
©2010 DENTSPLY Rinn. All rights reserved.
MK358 6.2010